

Cardiac
Markers

Coagulation
Markers

Diabetes
Mellitus

Inflammation

Thyroid
Function

Metabolic
Marker

Renal
Function

Tumor
Markers

Reproduction
/Fertility

Infectious
Disease



Getein
Biotech, Inc.

Stock Code: 603387

OPTIMIZED POINT-OF-CARE SOLUTION

MAKING TEST EASY

Getein 1100

Immunofluorescence Quantitative Analyzer



Getein 1100

Immunofluorescence Quantitative Analyzer



HIGHLY EFFICIENT & ACCURATE

Advanced fluorescence immunoassay

Multiple quality control



REAL-TIME AND RAPID TEST


One-step test

3-15 min/test

5 sec/test for multiple tests

OPERATION MODES


Inside Mode (single sample rapid test mode)

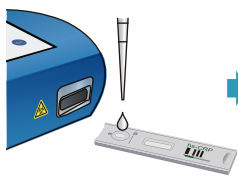

Sample Transfer



Test Card Insert

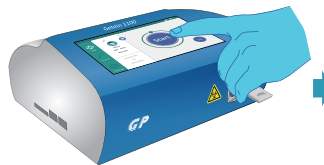

Click "Start" Icon




Result Show and Print


Quick mode (mass samples rapid test mode)



Sample Transfer

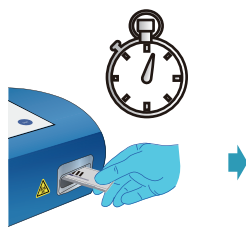

Timing the Reaction Manually

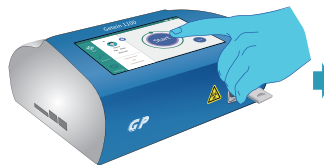

Click "Start" Icon



Result Show and Print



CONVENIENT OPERATION

RFID card calibration

Keyboard and mouse connectivity through USB port

Handwriting input available


Continuous test for 3 hours with optional lithium battery



USER-FRIENDLY INTERFACE

Android system

7-inch touch screen

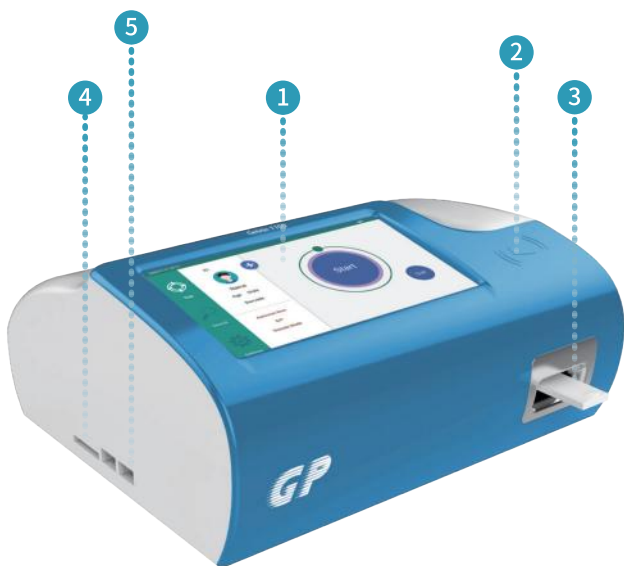

1 7-inch Touch Screen

3 Test Card Slot

5 USB Slot

2 SD Card Recognition Zone

4 SD Card Slot

6 Built-in Thermal Printer



PORTABLE DESIGN

Small in size: 261 × 241 × 115 mm

Light in weight: 2.0 kg



LARGE MEMORY

Up to 10,000 results storage capacity

TECHNICAL PARAMETERS

Methodology	Immunofluorescence	Screen	7-inch touch screen
Result	Quantitative	Power Supply	100-240 V~50 Hz/60 Hz, 60 VA
Sample Type	WB, Plasma, Serum, Urine, Stool, Nasal swab, Saliva, Capillary blood	Working Environment	Temperature: 10-35°C Relative humidity ≤ 70% Air pressure 70.0~106.0 kpa
Storage Capacity	10000 data	Dimensions	261 mm×241 mm×115 mm (D×W×H)
Language	English/Chinese/Spanish/Portuguese	Weight	2.0 kg

TEST ITEMS

Cat. #	TEST ITEMS	DISEASES	CUT-OFF VALUE	SAMPLE TYPES	MEASURING RANGE	SAMPLE VOLUME	REACTION TIME	QUALIFICATION
Cardiac Markers								
IF1001	cTnI	Myocardial infarction	0.10 ng/mL	S/P/WB	0.10-50.00 ng/mL	100 µL	10 min	NMPA CE
NEW IF1098	TnT	Myocardial infarction	14.0 pg/mL	S/P/WB	10.0-10000.0 pg/mL	100 µL	15 min	NMPA CE
IF1089	BNP	Heart failure	100.0 pg/mL	P/WB	5.0-5000.0 pg/mL	100 µL	10 min	NMPA CE
IF1002	NT-proBNP	Heart failure	300 pg/mL	S/P/WB	100-35000 pg/mL	100 µL	10 min	NMPA CE
IF1005	CK-MB/cTnI/Myo	Myocardial damage /infarction	CK-MB: 5.00 ng/mL cTnI: 0.10 ng/mL Myo: 70.0 ng/mL	S/P/WB	2.50-80.00 ng/mL 0.10-50.00 ng/mL 30.0-600.0 ng/mL	100 µL	10 min	NMPA CE
IF1012	CK-MB/cTnI	Myocardial damage /infarction	CK-MB: 5.00 ng/mL cTnI: 0.10 ng/mL	S/P/WB	2.50-80.00 ng/mL 0.10-50.00 ng/mL	100 µL	10 min	CE
IF1014	H-FABP	Myocardial damage	6.36 ng/mL	S/P/WB	1.00-120.00 ng/mL	100 µL	3 min	NMPA CE
IF1016	CK-MB/cTnI/H-FABP	Myocardial damage /infarction	CK-MB: 5.00 ng/mL cTnI: 0.10 ng/mL H-FABP: 6.36 ng/mL	S/P/WB	2.50-80.00 ng/mL 0.10-50.00 ng/mL 2.00-100.00 ng/mL	100 µL	10 min	NMPA CE
IF1018	CK-MB	Myocardial injury	5.00 ng/mL	S/P/WB	2.50-80.00 ng/mL	100 µL	10 min	CE
Coagulation Marker								
IF1006	D-Dimer	Venous thromboembolism	0.50 mg/L	P/WB	0.10-10.00 mg/L	100 µL	10 min	NMPA CE
Inflammation								
IF1003	hs-CRP+CRP	Cardiovascular inflammation /normal inflammation	3.0 mg/L 10.0 mg/L	S/P/WB/ Fingertip blood	0.5-200.0 mg/L	10 µL	3 min	NMPA CE
IF1007	PCT	Sepsis, bacterial infection	0.10 ng/mL	S/P/WB	0.05-50.00 ng/mL	100 µL	15 min	NMPA CE
IF1015	PCT/CRP	Sepsis, bacterial infection	PCT: 0.10 ng/mL CRP: 3.0 mg/L	S/P/WB	0.10-50.00 ng/mL 0.5-200.0 mg/L	100 µL	15 min	NMPA CE
IF1044	SAA	Bacterial/Virus infection	10.0 mg/L	S/P/WB/ Fingertip blood	5.0-200.0 mg/L	10 µL	5 min	NMPA CE
IF1090	SAA/CRP	Neonatal sepsis, Bacterial/virus infection	SAA: 10.0 mg/L CRP: 10.0 mg/L	S/P/WB/ Peripheral blood	5.0-200.0 mg/L 0.5-200.0 mg/L	10 µL	5 min	NMPA CE
IF1088	IL-6	Acute inflammation	7.0 pg/mL	S/P/WB/ Peripheral blood	1.5-4000.0 pg/mL	100 µL	15 min	NMPA CE
Renal Function								
IF1008	CysC	Acute and chronic renal diseases	0.51-1.09 mg/L	S/P/WB	0.50-10.00 mg/L	10 µL	3 min	NMPA CE
IF1009	mAlb	Diabetic nephropathy, hypertensive nephropathy	20.0 mg/L	Urine	10.0-200.0 mg/L	100 µL	3 min	NMPA CE
IF1010	NGAL	Acute kidney injury	Serum: 200.0 ng/mL Urine: 100.0 ng/mL	S/Urine	50.0-5000.0 ng/mL	10 µL	10 min	NMPA CE
IF1011	β ₂ -MG	Acute and chronic kidney diseases/tumours	0.80-3.00 mg/L	S/P/WB	0.50-20.00 mg/L	10 µL	3 min	NMPA CE
Diabetes Mellitus								
IF1017	HbA1c	Diabetes mellitus	3.80%-5.80%	WB	2.00%-14.00%	10 µL	5 min	NGSP NMPA IFCC CE
Metabolic Marker								
IF1031	25-OH-VD	Osteomalacia, osteoporosis	30.00-50.00 ng/mL	S/P	8.00-70.00 ng/mL	40 µL	15 min	NMPA CE
Thyroid Function								
IF1024	TSH	Thyroid malfunction	0.27-4.20 µIU/mL	S/P	0.10-50.00 µIU/mL	100 µL	15 min	NMPA CE
IF1022	T3	Hyperthyroidism, hypothyroidism	1.30-3.10 nmol/L	S/P	0.30-10.00 nmol/L	40 µL	15 min	NMPA CE
IF1023	T4	Hyperthyroidism, hypothyroidism	59.00-154.00 nmol/L	S/P	5.40-320.00 nmol/L	100 µL	15 min	NMPA CE
IF1067	fT3	Hyperthyroidism, hypothyroidism	3.10-6.80 pmol/L	S/P/WB	0.40-50.00 pmol/L	40 µL	15 min	CE
IF1068	fT4	Hyperthyroidism, hypothyroidism	12.00-22.00 pmol/L	S/P/WB	0.30-100.00 pmol/L	100 µL	15 min	CE

Cat. #	TEST ITEMS	DISEASES	CUT-OFF VALUE	SAMPLE TYPES	MEASURING RANGE	SAMPLE VOLUME	REACTION TIME	QUALIFICATION	
Reproduction/Fertility									
IF1013	HCG+β	Fertility	5.1 mIU/mL	S/P	5.0-100000.0 mIU/mL	100 μL	10 min	NMPA CE	
IF1055	LH	PCOS, infertility evaluation	Refer to User Manual	S/P	0.20-150.00 mIU/mL	100 μL	15 min	NMPA CE	
IF1056	FSH	PCOS, infertility evaluation and pituitary disorders	Refer to User Manual	S/P	0.20-150.00 mIU/mL	100 μL	15 min	NMPA CE	
IF1066	AMH	Fertility, PCOS, gonadal function, precocious/late puberty	Refer to User Manual	S/P	0.10-20.00 ng/mL	200 μL	15 min	CE	
IF1048	PRL	Infertility, gonadal disorders	Refer to User Manual	S/P	0.50-200.00 ng/mL	100 μL	15 min	NMPA CE	
IF1071	Prog	Infertility, evaluation of ovulation	Refer to User Manual	S/P	0.10-40.00 ng/mL	100 μL	15 min	CE	
NEW	IF1073	Testosterone	Female polycystic ovary syndrome, male testosterone insufficiency	Male: 1.75-7.81 ng/mL Female: 0.10-0.75 ng/mL	S/P	0.10-16.00 ng/mL	100 μL	15 min	CE
NEW	IF1074	E2	Ovarian function	Refer to User Manual	S/P	40.0-4800.0 pg/mL	100 μL	15 min	CE
Tumor Markers									
IF1053	tPSA	Prostate cancer	4.00 ng/mL	S/P	0.50-100.00 ng/mL	100 μL	15 min		
IF1072	fPSA	Prostate cancer	1.00 ng/mL	S/P	0.05-30.00 ng/mL	100 μL	10 min		
IF1050	AFP	Liver cancer, cancer of ovaries or testicles, etc.	7.0 ng/mL	S/P	2.0-500.0 ng/mL	100 μL	15 min	CE	
IF1051	CEA	Cancer marker: colon cancer etc.	4.7 ng/mL	S/P	2.0-500.0 ng/mL	100 μL	15 min	CE	
Infectious Disease									
IF1057	Anti-HCV	Hepatitis C	1.00 S/CO	S/P	1.00-20.00 S/CO	100 μL	15 min		
IF1058	Anti-TP	Syphilis	1.00 S/CO	S/P	1.00-50.00 S/CO	100 μL	15 min	CE	
IF1059	Anti-HIV	AIDS	1.00 S/CO	S/P	1.00-1000.00 S/CO	100 μL	15 min		
IF1064	HBsAg	Hepatitis B	1.00 IU/mL	S/P	1.00-100.00 IU/mL	100 μL	15 min		
NEW	IF1063	Anti-HBs	Hepatitis B	10.00 mIU/mL	S/P/WB	10.00-1000.00 mIU/mL	100 μL	15 min	
IF1084	2019-nCoV IgM/IgG	COVID-19	1.00 COI	S/P/WB		100 μL	10 min	CE	
NEW	IF1091	SARS-CoV-2 Antigen	COVID-19	1.00 COI	Nasal swab/Saliva	100 μL	15 min	CE	
NEW	IF1095	SARS-CoV-2 Neutralizing Antibody	COVID-19	Refer to User Manual	S/P/WB/ Fingertip blood	40 μL	15 min	CE	
IF1047	<i>H. pylori</i>	<i>H. pylori</i> infection	5.0 ng/mL	Stool	1.0-200.0 ng/mL	3 drops (about 100 μL)	10 min	CE	
NEW	IF1086	Influenza A/B	Respiratory viral infection	1.00 S/CO	Nasal swab	100 μL	15 min	CE	
NEW	IF1136	Dengue NS1 Ag	Dengue virus infection	1.00 S/CO	S/P/WB	0.50-50.00 S/CO	100 μL	15 min	CE
Specific Protein and Rheumatism									
NEW	IF1075	RF	Rheumatoid arthritis	15.9 IU/mL	S/P/WB	10.0-640.0 IU/mL	10 μL	10 min	CE
NEW	IF1076	ASO	Rheumatic fever, acute glomerulonephritis, group A streptococcal infection	408.0 IU/mL	S/P/WB	60.0-1370.0 IU/mL	10 μL	10 min	CE
NEW	IF1029	Anti-CCP	Rheumatoid arthritis	25.0 U/mL	S/P/WB	10.0-400.0 U/mL	10 μL	15 min	CE
Others									
NEW	IF1077	Ferritin	Anemia/tumors	Male: 30.00-400.00 ng/mL Female: 13.00-150.00 ng/mL	S/P	0.50-1000.00 ng/mL	10 μL	15 min	CE
NEW	IF1069	Total IgE	Allergic disorders	Refer to User Manual	S/P/WB	1.00-2000.00 IU/mL	100 μL	15 min	CE
NEW	IF1052	PG I/PG II	Atrophic gastritis, stomach cancer	PG I < 70.0 ng/mL PG I/PG II < 3.0 ng/mL	S/P	PG I: 1.0-200.0 ng/mL PG II: 1.0-100.0 ng/mL	100 μL	15 min	

Coming Soon: FOB, Folate...

GP Getein Biotech, Inc.

Add.: No.9 Bofu Road, Luhe District, Nanjing, 211505, China
 Tel.: +86-25-68568508/68568594
 Fax: +86-25-68568500
 E-mail: sales@getein.com.cn; overseas@getein.com.cn
 Web.: www.getein.com

ISO FSC CE NMPA NGSP IFCC IVD
ISO13485

